# LAMIDERM® Crème

## Samenstelling

Gezuiverd water, ethyleenglycolstearaat, paraffine dun vloeibaar, pentaandiol, stearinezuur, vaste paraffine, perhydroqualeene, avocado-olie, natriumalgiinaat, trolamine, cetylpalmitaat, kaliumsorbaat, parfum honella

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# LAMIDERM® Cream

## Composition

Purified water, ethylene glycol stearate, liquid paraffin, pentanediol, stearic acid, paraffin wax, perhydroqualene, avocado oil, sodium alginate, trolamine, cetyl palmitate, potassium sorbate, scent of honelia.

## Indications

- Cutaneous use
- On first or second degree burns, sunburn and non-infected wounds.
- In the case of reddened skin due to radiotherapy.

## Instructions for use

### Tube opening

At the first use, pierce the tube opercula with top of the cap.

### - First degree burns and sunburn

Apply Lamiderm in a thick layer and allow it to penetrate without rubbing it in. Repeat this process two or four times a day.

Apply a thick layer

Let it penetrate without rubbing it in

### - Second degree burns and non-infected wounds

After cleansing the burn or non-infected wound apply a thick layer of Lamiderm and repeat this several times each day.

cleansing

Apply a thick layer

Cover with a moistened compress

If required, cover with a moistened compress. Do not use a dry absorbent dressing.

### - Reddened skin due to radiotherapy

Apply Lamiderm two or three times a day, according to the physician recommendations, and let it penetrate with a light massage.

Apply a thick layer

Let it penetrate with a light massage

## Precautions for use

Do not use:

- If the wound is bleeding
- If the wound is infected.

This medical device is not a sunscreen.

Do not use in case of hypersensitivity to any component of the product.

Keep out of the reach of children.

Store away from heat and in the carton box.

Do not exceed the use-by date clearly indicated on the packaging and the tube.

## Characteristics

Tube of 15 ml, 60 ml or 140 ml.

Revision date: 07/2019

Distributor: Qualiphar N.V./S.A., Rijksweg 9, B-2880 Bornem

Manufacturer: Gifrer Barbezat

8 -10 rue Paul Bert

69150 DECINES CHARPIEU – FRANCE